savvycheck

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www.savvyondiagnositics.com

HOW TO READ THE TEST RESULTS

The pictures provided are meant to help you in interpreting your results and are meant only as an example.

Within 3-10 minutes, you should see a pink line in the Control Window (small round window). This Control Line indicates that the test is complete and you did the test correctly.

						
Day 1 Negative	Day 2 Negative	Day 3 Surge Positive	Day 4 Negative	Day 5 Negative	Day 6 Negative	Day 7 Negative

NEGATIVE TEST RESULT

If the Test Line in the square test window is lighter than the Control Line (or if there is no Test Line at all), the result is negative. No surge has been detected. You should continue to test daily.

POSITIVE TEST RESULT

If the Test Line in the square window is darker than or equal to the Control Line in the round window, then the test result is positive and indicates an LH surge. You are likely to ovulate in the next 24-36 hours. Once you have detected your surge, you can stop testing for this cycle.

INVALID TEST RESULT

If no Control Line appears, the test is considered invalid.

Do not read the result after 10 minutes. Do not reuse the test.

OVULATION AND FERTILITY

After you have noticed the LH surge in your urine, you are likely to ovulate in the next 1-2 days. Ovulation normally takes place once a month. During ovulation, an egg is released by one of the ovaries. In the next 24 hours, the egg travels through the fallopian tube to the uterus.

Fertilization of the egg by a male sperm takes place in the fallopian tube. Therefore, a healthy ovulating woman is most fertile during her ovulation period (1-2 days after LH surge). A male sperm can usually stay active to fertilize an egg for a maximum of 72 hours.

Q & A – FREQUENTLY ASKED QUESTIONS

Q. What time of the day should I take the test?

A. You can take the test any time of the day, but for the best results you should test about the same time each day. You should avoid drinking too much liquid for at least 2 hours before you test. This may dilute the amount of LH in your urine. The best time for testing is between 10 AM and 8 PM.

Q. How long does it take to perform the test?

A. It is important to hold the absorbent tip in your urine stream for at least 10 seconds and then wait at least 3 minutes before reading your results.

Q. Do I need to use all 7 tests?

A. You can stop testing when you detect your LH surge and save the remaining tests to use next month, if necessary.

Q. What if no line appeared in either window?

A. It could be because you didn’t hold the absorbent tip in your urine stream long enough or you didn’t place the test on a clean, flat surface with the windows facing upward. If this happens, you should throw the test away and repeat the test with a new Ovulation Predictor.

Q. What if I am unsure of my test results?

A. The test is so easy to use. The built-in-Control Line in the small round window means that you did the test right. If you do not see a Control Line in the small round window, the test did not work. You should repeat the test using another Ovulation Predictor. Make sure to follow the instructions carefully. If you want to confirm that your LH surge has occurred, test again about 4 hours later and remember not to urinate during this time. The line in the test window (the large square window) should be as dark or darker than the Control Line.

Q. I have recently stopped using hormonal birth control. Will this affect the results?

A. After using the pill, your cycle can be irregular and may take some time to stabilize again. In this case, you may need to use more than one Ovulation Predictor kit each cycle. (You may want to wait until you have had 2 normal periods before starting the test. Follow your doctor’s advice as to when you should try to conceive.)

Q. How accurate is the Ovulation Predictor?

A. The accuracy of the test is 99% when compared to other similar tests on the market.

Q. I’ve been using the basal body temperature (BBT) method. Does the Ovulation Predictor take its place?

A. The Ovulation Predictor indicates that ovulation is about to occur. The BBT method indicates a change in body temperature, which is after ovulation has occurred. Consult with your doctor for more information.

Q. I’ve used all 7 tests as instructed, but I have not yet detected the LH surge, what do I do?

A. The 7 tests contained in the Ovulation predictor kit are usually enough for women with regular cycles (if your cycle varies by fewer than 3 days). If your cycle is irregular, you may need to use additional Ovulation Predictors to predict the time of peak fertility.

For external diagnostic use only. (Not to be used internally)

Manufacturer: Savyon Diagnostics Ltd.,

3 Habosem Street, Ashdod, 7741003, Israel.

Exclusive Distributor: Gamidor Diagnostics LTD

32 Shaham St. Petah Tikva 4951727

MOH Registration certificate Number - 26520009

1800-071-081 | www.savvyondiagnositics.com

savvycheck

ONE STEP OVULATION PREDICTOR

INSTRUCTIONS

Please read all instructions carefully. For accurate results - follow directions carefully. Do not take the test device out of the foil pouch until you are ready to use it.

INTENDED USE

The Ovulation Predictor is a fast, easy to use midstream test for detection of luteinizing hormone (LH) in urine. The Ovulation Predictor helps you to identify the time you are likely to ovulate.

The tests help you monitor the amount of LH in your urine. Normally, during each cycle, the level of LH suddenly increases (this is called the LH surge) causing the release of an egg from the ovary 24-36 hours later. This process is called ovulation. During ovulation is when you are most likely to become pregnant.

IMPORTANT

- This test is not to be used for contraception (birth control) or as an aid to contraception.
- Do not take the test out of the foil pouch until you are ready to use it.
- The LH surge and ovulation may not occur in all cycles.
- Some medical conditions such as pregnancy, postpartum, post abortion or ovarian cysts may give an inaccurate result.
- False high LH may occur in untreated hyperthyroid patients.
- If you are being treated by your doctor with Clomiphene Citrate or any medication that may contain hCG or LH, consult your doctor before using this test.
- The results of this test may not be valid if you are taking hormone-based birth control.
- Do not use after the expiration date printed on the package.
- You should not have urinated for at least 4 hours prior to testing.
- Keep out of childrens’ reach
- Dispose of the device in the regular trash after use.
- Do not drink approximately 2 hours prior to urine collection.
- For external use only
- Store at 2-30 °C

SPECIMEN COLLECTION

Decide what time of the day you want to do the test. The best time is between 10 AM and 8 PM. For best results, perform the test at about the same time each day. Do not do this test on the first morning urine.

WHEN TO TEST

Determine the length of your menstrual cycle.

The length of your cycle is from the first day of your period (menstrual bleeding) to the last day before your next period starts.

If your cycle length is irregular, that is, if it varies by more than a few days each month, count your shortest cycle in the last six months.

Use the chart to plan the day you should begin testing.

Cycle length (Days)	21	22	23	24	25	26	27	28	29	30	31	32	33	34	35	36	37	38
Day of cycle to begin testing	6	6	7	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21

Begin testing. Continue testing until you have seen your LH surge. If after 7 days you have not seen your surge, you may need to buy another box of Ovulation Predictor and continue testing.

HOW TO USE OVULATION PREDICTOR

Kit Contents:

- Seven (7) individually wrapped Ovulation test devices.
- Instructions.

How to do the test:

- Using the chart provided, decide when you need to start testing.
- Make sure you have an available clock.
- When you are ready to test, remove one (1) of the test devices from the box.
- Open the foil pouch by tearing at the notch. Do not take the test device out of the foil pouch until you are ready to use it.
- Remove the protective cap covering the absorbent tip.
- Hold the test absorbent tip in your urine stream for 10 seconds, being sure to wet the tip. Alternatively, you may collect your urine in a clean container and dip the tip into the urine for 10 seconds.
- After removing the test device from the urine, replace the cap and lay the test on a flat surface with the Test/Control windows facing up.
- Wait at least 3 minutes before reading results. Do not read the results after 10 minutes

Note: As the test begins to work, you may see a pink/purple color moving across the windows. This is normal.