## EARLY PREGNANCY TEST


### RAPID PREGNANCY TEST

- Simple and easy to use pregnancy test.
- Over 99% accurate
- With results in just 3-10 minutes

Read the instructions for use carefully and completely before beginning the test.

- The test can be used until the expiration date printed on the package.
- The test should be stored at room temperature (2-30°C).
- If you have any questions regarding your results, please refer to the Questions and Answers section.


### THE DESIGN OF THE TEST STICK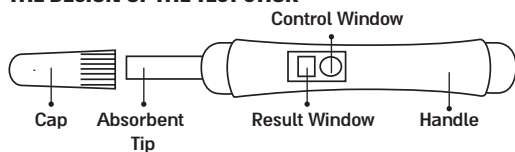

### INSTRUCTIONS FOR USE:

1. The test is hermetically sealed; open the pouch when you are ready to begin the test.
2. Take out the test stick and remove the protective cap covering the absorbent tip.
3. When urinating, hold the test stick with the absorbent tip in the urine stream for a minimum of TEN SECONDS, until the entire tip is wet. Alternatively, you may collect your urine in a clean container and dip the tip into the urine for 10 seconds. Refrain from urinating on the test windows (control window and result window - the clear portion of the test stick.)
4. Replace the cap on the tip and lay the test stick on a flat surface with the windows facing up. You will notice a pink color moving across the windows - this is normal.
5. Read the results in the square result window, after 3-10 minutes. Do not read the results after 10 minutes.

### READING THE RESULTS

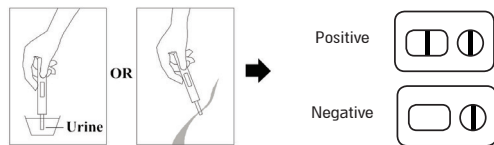### POSITIVE RESULTS - PREGNANT

2 pink lines will appear, one in each of the 2 windows. It does not matter which line is darker.

### NEGATIVE RESULTS - NOT PREGNANT

Only 1 line will appear, in the round control window.

### Note:

- If no line appears in the round control window, the test is invalid and must be done again.

### TEST LIMITATIONS:

- The test device is NOT reusable.
- The test only works if used carefully according to the instructions.
- Certain health conditions, such as ectopic pregnancy (pregnancy outside the uterus) or ovarian cyst, can give false results.
- Some fertility drugs containing the hormone hCG, could alter the test results. Conducting the test after recently taking the drugs could give false positive pregnancy results.
- Oral contraceptives, pain killers, antibiotics, and other drugs do not affect the results of the test.
- The sensitivity of the test is > 99%, but it should be confirmed by your doctor and after performing a clinical test.

### Q & A – FREQUENTLY ASKED QUESTIONS

**Q.** How does the test work?

**A.** The test is based on detection of small amounts of the pregnancy hormone hCG. This hormone is produced in an increasing amount during the first part of pregnancy. The test can be conducted from the first day of a missed period.

**Q.** Is it necessary to test in the morning?

**A.** NO. You can test any time of the day. However, the first morning urine contains the highest amounts of hCG and therefore may give more reliable results.

**Q.** What if the 2 lines in the test windows are not identical?

**A.** The darkness of the lines is not significant. One line could be darker than the other.

**Q.** What is the purpose of the line in the ROUND control window?

**A.** The appearance of this line indicates the test has been performed properly. If NO line appears in ROUND window, the test should be repeated with a new device, carefully following the instructions for use.

**Q.** How long until the results appear?

**A.** Results can appear in as little as 1-3 minutes. In some cases, the results will appear after 3 minutes. However, do not read results after 10 minutes.

**Q.** Can I use a urine sample that is collected in a container?

**A.** YES, As long as the urine is collected in a clean, dry container. To perform the test in this manner, simply immerse the absorbent tip into the urine for AT LEAST 10 seconds. Then continue the test, following the instructions for use.

**Q.** How sensitive is the test?

**A.** The Savvycheck Pregnancy Test is an accurate and sensitive self-test. (the test detects 10 mIU/mL or greater of the hCG hormone in urine)

**Q.** What should I do if the test results are negative?

**A.** If your menstrual period does not begin by the end of the week it was scheduled to begin, do another pregnancy test. If the results are still negative, consult your physician.

**Q.** How soon can I take the test?

**A.** You can take the test from the first day when you should have begun your menstrual period.

**For external diagnostic use only. (Not to be used internally)**

Manufacturer: Savyon Diagnostics Ltd., 3 Habosem Street, Ashdod, 7761003 Israel.  
Exclusive Distributor: Gamidor Diagnostics LTD  
32 Shaham St. Petah Tikva 4951727  
1800-071-081

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